



71 0806 TR Contact Alarm

Instructions for Operation, Use & Setup Options

Warning: Before using the AliMed TR Contact Alarm, read entire instruction sheet and save for future reference. Failure to do so may result in injury or death to a person in your care.

Application

AliMed's TR Contact Alarm is used with an alarmed fall mat or floor sensor pad to alert staff that a resident has stepped onto the pad or has fallen out of bed. When sensor is activated, an 85dB or 100dB alarm tone sounds, notifying the caregiver. Alarm activates immediately when pressure is applied to pad.

Operating Instructions

Your new AliMed TR Contact Alarm includes four AA batteries. Alarm unit supports floor mats and floor sensor devices. Do not use with patients in bed or in a chair.

To use:

- Battery installation: Open battery compartment by loosening screw on base of the cover. Install batteries accordingly. Ensure contacts are lined up.
- Remove the temporary Yellow reset button identification sticker on the front of the unit.
- Slide the switch at the top of the unit to the "LOAD" position.
- Insert the sensor cable into the slot that has been revealed by moving the slide switch.
- Move the slide switch into the "LOCK" position.
- *Test Alarm by attaching a sensor device to alarm. Put pressure on pad. Alarm should sound. The alarm is now ON, locked, and ready for use. The lock position incorporates a cable strain relief mechanism to reduce tensile strain on the cable and fitting.
- Molded reset buttons on each side must be pressed simultaneously to reset the alarm.
- If using mesh bag (#75303) to attach alarm to a chair, bed, or table leg, place alarm in bag and close zipper. Alarm cord should exit bag at zipper opening. Use strap on bag to hang the unit, or secure to chair or bed using hook and loop straps.
- Be sure to detach the sensor cord from the alarm unit when moving the contact alarm or floor sensor pad to avoid damaging either the alarm or sensor pad.

NOTE: Alarm cannot be turned off when in use. To store alarm, remove cable and slide switch to IDLE position to conserve batteries. SILENT MODE only for use with wireless remote systems.

***Test Alarm daily to ensure patient safety**

Customized Setup Options

Loosen the silver screw on the base of the case sufficiently to remove the case lid.

Two set-up options are available:

1. Alert: chime, alarm, or silent—silent setting can be used with the nurse call jack connection. This allows the alarm to only sound at nurse call station.
2. Volume: 85 dB or 100 dB—Set to 85 dB for a quieter sounding alarm. Set to 100 dB for a louder sounding alarm if there are other noise-generating devices present.

Limited Six-Month Warranty

Manufacturer guarantees AliMed TR Contact Alarm for a period of six months from date of purchase. Warranty is void if unit has been opened, damaged, or submerged.


Manufactured for:




Dedham, MA 02026-9135

Order Toll Free 800-225-2610 • 781-329-2900 • Fax 800-437-2966

©2009 AliMed® inc. All rights reserved.

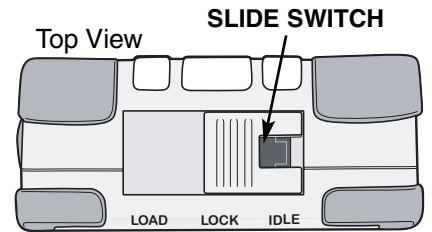


ONLY USE SILENT MODE WHEN CONNECTED TO NURSE SYSTEM OR WIRELESS CONVERT ALERT.

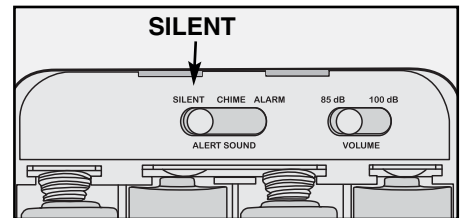

STATUS light will blink once when turned on. It will beep and blink when activated or reset.


ALERT light flashes continuously while alarm sounds.

LOW BATTERY light indicates that batteries must be replaced if blinking.

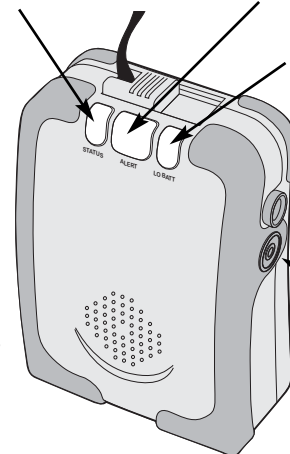


Nurse call jack socket

Molded reset buttons (one on each side).

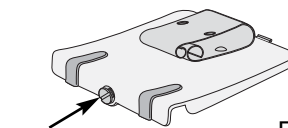

Captive screw won't get lost.

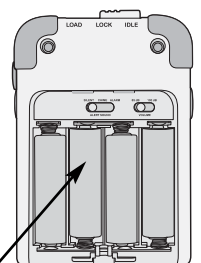

Four AA batteries (installed)

Reorder Information

- #71 0806 AliMed TR Contact Alarm
- #71 0807 2" Alarm Fall Mat w/TR Contact Alarm
- #71 0809 4" Alarm Fall Mat w/TR Contact Alarm