




# AliMed® Universal Alarm Strap

## Instructions For Use:

**Caution: We strongly recommend you familiarize yourself with the components and installation technique before attempting to install on a patient.**

**Read entire instruction sheet before using this Universal Strap and save for future reference. Failure to do so may result in injury or even death to a person in your care.**

1. Remove strap from box, remove rubber band, and unroll strap. Keeping **Blue** sensor connected, strap through loop.
2. Place on bed with label facing upward and **White** and **Red** loops on either side of the bed. Place looped section in the middle of the bed, with the **Blue** sensor facing away from the mattress surface (FIG. A).
3. Secure the end with the **Red** loop to the bed frame, out of patient's reach, using the hook-and-loop fastener (FIG. B2).

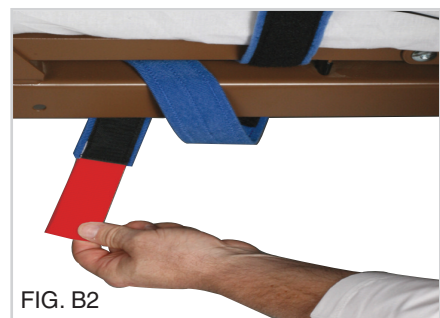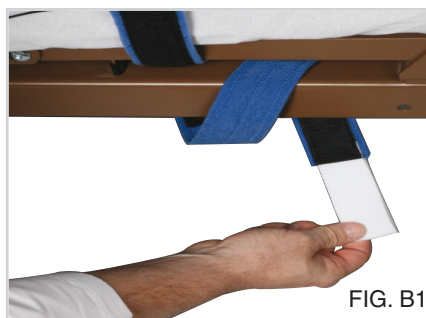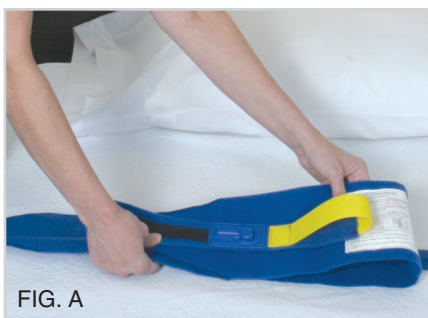

**For beds with movable frames:** Secure the end with the **Red** loop to the movable part of the frame. **Do NOT attach to fixed part of frame on movable frame beds.**  
**For beds with stationary frames:** Secure the end with the **Red** loop directly to the bed frame.



4. Undo the **Blue** connectors to open strap. Have patient lie in the middle of bed with strap open. Adjust belt so it fits patient securely and **Yellow** handle is located in the middle of the patient's abdomen (FIG. C).
5. Fasten **Blue** connectors by aligning opening to blue magnetic fastener.
6. Secure the end of the strap with the **White** loop to the other side of the bed frame, using the hook-and-loop fastener. To keep the **White** loop out of patient's reach, you may need to wrap this strap around the bed frame again before attaching hook-and-loop fastener.
7. Connect the cable to the alarm (FIG. D). **Note: ALWAYS TEST THE ALARM** by lifting the **Yellow** E-Z Release Handle. This yellow handle must always be outward on the front of the patient and within the patient's reach.



**This product is not intended to be used as a restraint.**

AliMed® Universal Alarm Strap allows patients the freedom to roll or sit up in bed, yet still alerts caregivers when the patient attempts to *leave* the bed. **Intended for bed use only.**

**Contraindications:**

- Patients who lack the dexterity or ability to release hook-and-loop type closures
- Patients who are agitated, aggressive, combative, violent, or suicidal
- Patients who cannot sit up unassisted
- Patients who are not lucid
- Patients with postsurgical incisions, tubes, lines, or other monitoring equipment
- Patients under 18 years of age (without consent of guardian)
- Patients with pacemakers

Never use this product as a seatbelt in a chair or motor vehicle.

**Constant patient monitoring is required, per your facility's policy.**

**HAND-WASH ONLY**

Be prepared to intervene at the first sign of danger.

Before using this product, read enclosed instructions carefully to ensure proper usage.

For additional copies of Instruction Sheet #6378 call **800-225-2610**.

**Important Note:**

This product may be supplied with any of five AliMed alarms, the TR2 Tamper-Resistant (#79990), the TR2 w/silent setting (#710985), the TR2+ (#711887), the IQ Duo Plus (#711290), or the VOICE Alarm (#2032). For correct operation, please read and follow the appropriate instructions for the respective alarm unit.

Use of any alarm except the AliMed alarms identified above, in conjunction with the Universal Alarm Strap System, may lead to improper function of the product. Use of the Universal Alarm Strap with any other alarm except those identified will void all warranties and is not recommended.

**Do not use if product is damaged or incomplete.**

**Note: This product will not prevent falls, it is intended for use as a warning device.**

**Reorder Information:**

**#ZX70089** Replacement Strap, Standard, 125"L

**Systems:**

**#ZX70088** Standard Strap w/TR2 Alarm™  
**#ZX71 1359** Standard Strap w/IQ Duo Plus Alarm™  
**#ZX72121** Standard Strap w/VOICE Alarm™

Distributed by:



**AliMed® inc.**

297 High St., Dedham, MA 02026-9135  
www.AliMed.com 800-225-2610